

Etch 'N' Seal®

Enamel and Dentin Etchant



Etch 'N' Seal is a special formulation of phosphoric acid and aluminum oxalate. This product allows you to etch enamel and protect dentin from possible over-etching. Oxalates are also shown to inhibit demineralization and secondary caries. Etch 'N' Seal etches enamel and dentin simultaneously to maximize bond strength. For complete sealing of dentin tubules and optimal bond strength to dentin and enamel, we recommend Tenure® Multi-Purpose Bonding System.



DIRECTIONS

1. Clean all tooth surfaces thoroughly and remove all caries.
2. Apply Etch 'N' Seal to the enamel and dentin for 10-15 seconds.
3. Rinse for 10 seconds and gently blot dry or air-dry.
4. Sealing tubules: We recommend Tenure Multi-Purpose Bonding System (Kit No. 031146000).


STORAGE

Do **not** expose to temperatures exceeding 77° F (25° C).

Do **not** freeze.

Cold storage is not required.

SAFETY DATA SHEETS AVAILABLE AT denmat.com

 Den-Mat Holdings, LLC
1017 W. Central Avenue
Lompoc, CA 93436 USA



MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany

Symbols Glossary available at:
denmat.com/symbols

